



ACCU-Flow *Connect* ACCU-SET *Micro*

Infusion set with Y-Injection Port & Micro Drip

INSTRUCTIONS FOR USE:

- Remove the outer cap of the spike and disinfect the top.
- Close the regulator firmly.
- Insert the spike at the top of the container to its full length, to the container upside down and hang it at suitable height.
- Squeeze the drip chamber and adjust the fluid level till half-filled to prevent air bubbles from entering the fluid path.
- Remove the protective cover & gently release the regulator in order to let the solution flow to pass until all air bubbles in the tube are removed.
- Close the regulator.
- Perform vein puncture and release the regulator to achieve the desired flow rate.
- After use, dispose off the device as per applicable local, state & country laws and regulations.



EXPLANATION OF SYMBOLS:

	Caution		Medical device		Non-pyrogenic
	Manufacturer		Unique device identifier		Sterilized by irradiation
	Country of manufacture		Individual sterile barrier system with outer protective packaging		Handle with care
	Do not resterilize		Authorized representative in the European Community		Maximum stacking
	See instructions for use.		Biological risks		Prescription use
	Keep away from sunlight		Catalog number		Model number
	Do not reuse		Batch code		Unique sterile barrier system
	Fluid path		Use by date		Date of manufacture
	This side up		Temperature limit		Fragile, handle with care
	Do not use hook		Do not use if package is damaged and see instructions for use.		Keep dry

SPM Medicare Pvt. Ltd.

(An EN ISO 13485 Certified Company)

Corp. & Mfg. Address:

B-40, Phase-II, Noida, Gautam Buddha Nagar-201305, U.P., INDIA

Customer Care: +91-8407070718 E-mail : info@sppmedicare.com

Website : www.sppmedicare.com