



ACCU Pedia[®]

Measured Volume Administration Set

INSTRUCTIONS FOR USE:

- Remove the outer cap of the container and disinfect top.
- Firmly close the regulator.
- Remove spike cover and insert the spike at top of the container to its full length.
- Squeeze the drip chamber until it is filled up to the mark. This prevents air Bubbles from entering the fluid path.
- Remove the protective cover of needle. Gently release the regulator in order to let the blood flow, keeping the needle above the container level. Gradually lower the needle in such a way that no air bubbles is left on the tube.
- Check the graduation mark.
- Close the regulator.
- Perform vein puncture and release the regulator to achieve the desired flow rate
- Use Y-injection site for extra medicament.
- Dispose the device after use according to local or national law for medical waste.



EXPLANATION OF SYMBOLS:

	Caution		Medical device		Non-pyrogenic
	Manufacturer		Unique device identifier		Sterilized by irradiation
	Country of manufacture		Individual sterile barrier system with outer protective packaging		Handle with care
	Do not resterilize		Authorized representative in the European Community		Maximum stacking
	See instructions for use.		Biological risks		Prescription use
	Keep away from sunlight		Catalog number		Model number
	Do not reuse		Batch code		Unique sterile barrier system
	Fluid path		Use by date		Date of manufacture
	This side up		Temperature limit		Fragile, handle with care
	Do not use hook		Do not use if package is damaged and see instructions for use.		Keep dry

SPM Medicare Pvt. Ltd.

(An EN ISO 13485 Certified Company)

Corp. & Mfg. Address:

B-40, Phase-II, Noida, Gautam Buddha Nagar-201305, U.P., INDIA

Customer Care: +91-8407070718 E-mail : info@sppmedicare.com

Website : www.sppmedicare.com