
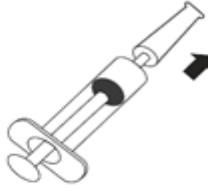
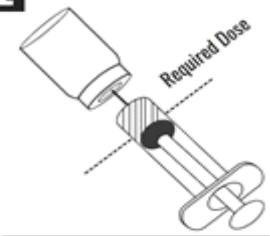
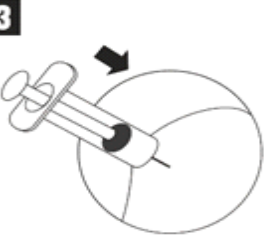


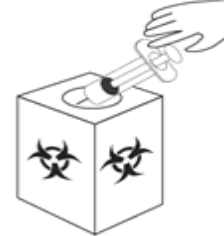

CoVax-Jet[®]

AUTO-DISABLE (AD)

INSTRUCTIONS FOR USE:

- 1**

 - Remove the syringe from its plastic wrapping by tearing the pouch.
 - Take off the needle cap without touching the needle.
- 2**

 - Insert the needle in the vaccine vial – its tip should be in the lowest part or bottom of the vial.
 - Pull the plunger back to fill the required dose.
 - Remove the needle from the vial. To remove air bubbles, hold the syringe upright and tap the barrel. Then carefully push the plunger to the volume mark.
 - Proceed with the injection at the appropriate site.
- 3**

 - Push the plunger forward and inject the vaccine. At the beginning or just at the end of the injection, the plunger will automatically lock so the syringe cannot be reused.
 - Do not recap the needle after use.
 - Destroy after single use.
- 4**

 - Dispose off the needle and syringe in a safety box, which is a leak-proof, puncture resistant container for sharps waste or as per applicable laws.



EXPLANATION OF SYMBOLS:

- | | | | | | |
|--|---------------------------|--|---|--|-------------------------------|
| | Caution | | Medical device | | Non-pyrogenic |
| | Manufacturer | | Unique device identifier | | Sterilized by irradiation |
| | Country of manufacture | | Individual sterile barrier system with outer protective packaging | | Handle with care |
| | Do not resterilize | | Authorized representative in the European Community | | Maximum stacking |
| | See instructions for use. | | Biological risks | | Prescription use |
| | Keep away from sunlight | | Catalog number | | Model number |
| | Do not reuse | | Batch code | | Unique sterile barrier system |
| | Fluid path | | Use by date | | Date of manufacture |
| | This side up | | Temperature limit | | Fragile, handle with care |
| | Do not use hook | | Do not use if package is damaged and see instructions for use. | | Keep dry |

SPM Medicare Pvt. Ltd.

(An EN ISO 13485 Certified Company)

Corp. & Mfg. Address:

B-40, Phase-II, Noida, Gautam Buddha Nagar-201305, U.P., INDIA

Customer Care: +91-8407070718 E-mail : info@sppmedicare.com

Website : www.sppmedicare.com