




ACCU-SET *Elite*®

Infusion Set with latex bulb

INSTRUCTIONS FOR USE:

- Peel open the pouch & take out the infusion set.
- Remove the protector cap of from piercing spike.
- Insert the piercing spike to its full length into the top of the container.
- Open air vent of piercing spike.
- Close flow regulator. Squeeze drip chamber till it is half filled.
- Open flow regulator & allow solution to pass till all air bubbles in the tube are removed.
- Close the Flow regulator
- Remove the priming stop cap from luer lock connector.
- Connect the set to intended device and regulate flow by using flow regulator.
- Use Y-injection site for extra medicament.
- For intermittent use put the prime stop cap back in luer connector.

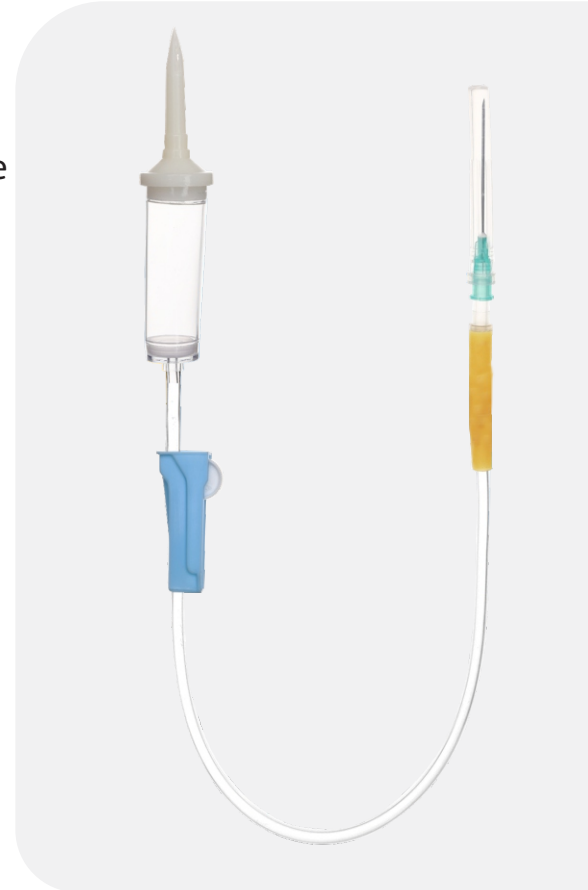

EXPLANATION OF SYMBOLS:

| | | | | | |
|--|---------------------------|--|---|--|-------------------------------|
| | Caution | | Medical device | | Non-pyrogenic |
| | Manufacturer | | Unique device identifier | | Sterilized by irradiation |
| | Country of manufacture | | Individual sterile barrier system with outer protective packaging | | Handle with care |
| | Do not resterilize | | Authorized representative in the European Community | | Maximum stacking |
| | See instructions for use. | | Biological risks | | Prescription use |
| | Keep away from sunlight | | Catalog number | | Model number |
| | Do not reuse | | Batch code | | Unique sterile barrier system |
| | Fluid path | | Use by date | | Date of manufacture |
| | This side up | | Temperature limit | | Fragile, handle with care |
| | Do not use hook | | Do not use if package is damaged and see instructions for use. | | Keep dry |

SPM Medicare Pvt. Ltd.

(An EN ISO 13485 Certified Company)

Corp. & Mfg. Address:

B-40, Phase-II, Noida, Gautam Buddha Nagar-201305, U.P., INDIA

Customer Care: +91-8407070718 E-mail : info@sppmedicare.com

Website : www.sppmedicare.com