



ACCU-FLOW™ *Airsafe Elite*

Infusion Set

INSTRUCTIONS FOR USE:

- Peel open the pouch & take out the infusion set.
- Remove the protector cap of from piercing spike.
- Insert the piercing spike to its full length into the top of the container.
- Open air vent of piercing spike.
- Close flow regulator. Squeeze drip chamber till it is half filled.
- Open flow regulator & allow solution to pass till all air bubbles in the tube are removed.
- Close the Flow regulator
- Remove the priming stop cap from luer lock connector.
- Connect the set to intended device and regulate flow by using flow regulator.
- Use Y-injection site for extra medicament.
- For intermittent use put the prime stop cap back in luer connector.



EXPLANATION OF SYMBOLS:

	Caution		Medical device		Non-pyrogenic
	Manufacturer		Unique device identifier		Sterilized by irradiation
	Country of manufacture		Individual sterile barrier system with outer protective packaging		Handle with care
	Do not resterilize		Authorized representative in the European Community		Maximum stacking
	See instructions for use.		Biological risks		Prescription use
	Keep away from sunlight		Catalog number		Model number
	Do not reuse		Batch code		Unique sterile barrier system
	Fluid path		Use by date		Date of manufacture
	This side up		Temperature limit		Fragile, handle with care
	Do not use hook		Do not use if package is damaged and see instructions for use.		Keep dry

SPM Medicare Pvt. Ltd.

(An EN ISO 13485 Certified Company)

Corp. & Mfg. Address:

B-40, Phase-II, Noida, Gautam Buddha Nagar-201305, U.P., INDIA

Customer Care: +91-8407070718 E-mail : info@spmmedicare.com

Website : www.spmmedicare.com