



# ACCU-FLOW Dial

Infusion Set with flow regulator & Y-Injection site

## INSTRUCTIONS FOR USE:

- Check the packing carefully, if packing is found damaged, torn or pierced, discard the piece.
- Tear/Peel open the pouch and take out the device aseptically.
- Remove the outer cap of the spike & close the regulator firmly.
- Insert the spike at the top of the container to its full length to the container upside down and hang it at suitable height
- Squeeze the drip chamber till it is half filled
- Open flow regulator & allow solution to pass until all air bubbles in the tube are removed
- Close the flow regulator
- Connect the set to intended device and regulate desired flow rate by using flow regulator
- After use, dispose off the device as per applicable local, state and country laws and regulations.



## EXPLANATION OF SYMBOLS:

	Caution		Medical device		Non-pyrogenic
	Manufacturer		Unique device identifier		Sterilized by irradiation
	Country of manufacture		Individual sterile barrier system with outer protective packaging		Handle with care
	Do not resterilize		Authorized representative in the European Community		Maximum stacking
	See instructions for use.		Biological risks		Prescription use
	Keep away from sunlight		Catalog number		Model number
	Do not reuse		Batch code		Unique sterile barrier system
	Fluid path		Use by date		Date of manufacture
	This side up		Temperature limit		Fragile, handle with care
	Do not use hook		Do not use if package is damaged and see instructions for use.		Keep dry

## SPM Medicare Pvt. Ltd.

(An EN ISO 13485 Certified Company)

Corp. & Mfg. Address:

B-40, Phase-II, Noida, Gautam Buddha Nagar-201305, U.P., INDIA

Customer Care: +91-8407070718 E-mail : info@sppmedicare.com

Website : www.sppmedicare.com