



# ACCU-FLOW™ UV Shield

## Light Resistant Infusion Set

### INSTRUCTIONS FOR USE:

- Carefully inspect the pack, don't use if the pack is open or set was damaged. Tear/Peel-open the pouch and take out the device aseptically.
- Remove the spike cap and Insert the spike to its full length into IV bag port. Open air vent of the spike.
- Close the Flow Regulator. Squeeze the drip chamber till it is half full.
- Open the clamp and allow fluid to fill the transfer set. Close the clamp.
- Open the Y connector cap and inject the drug through Y-port, flush a small amount of fluid back into the bag, then disconnect the syringe.
- Lock the luer tip of the properly prepared transfer set into one of the check valves/needle-free connectors of the infusion set.
- Perform the drug infusion by opening the clamp of the transfer set.
- Close the clamp of the transfer set when the infusion is finished.
- Dispose of the set without separating its parts, according to the local regulations for medical waste.



### EXPLANATION OF SYMBOLS:

	Caution		Medical device		Non-pyrogenic
	Manufacturer		Unique device identifier		Sterilized by irradiation
	Country of manufacture		Individual sterile barrier system with outer protective packaging		Handle with care
	Do not resterilize		Authorized representative in the European Community		Maximum stacking
	See instructions for use.		Biological risks		Prescription use
	Keep away from sunlight		Catalog number		Model number
	Do not reuse		Batch code		Unique sterile barrier system
	Fluid path		Use by date		Date of manufacture
	This side up		Temperature limit		Fragile, handle with care
	Do not use hook		Do not use if package is damaged and see instructions for use.		Keep dry

## SPM Medicare Pvt. Ltd.

(An EN ISO 13485 Certified Company)

Corp. & Mfg. Address:

B-40, Phase-II, Noida, Gautam Buddha Nagar-201305, U.P., INDIA

Customer Care: +91-8407070718 E-mail : info@sppmedicare.com

Website : www.sppmedicare.com