



# ACCU-SHOT™ EASY-JET™ Insulin Syringe

## INSTRUCTIONS FOR USE:

<p><b>1</b></p> <ul style="list-style-type: none"> <li>Remove upper cap</li> </ul>	<p><b>2</b></p> <ul style="list-style-type: none"> <li>Insert syringe into vial</li> <li>Pull back upto required dose</li> </ul>
<p><b>3</b></p> <ul style="list-style-type: none"> <li>Insert the needle through the skin carefully</li> <li>Push the insulin in by pressing the plunger down</li> </ul>	<p><b>4</b></p> <ul style="list-style-type: none"> <li>Dispose off the used insulin syringe &amp; needle</li> </ul>



## EXPLANATION OF SYMBOLS:

	Caution		Medical device		Non-pyrogenic
	Manufacturer		Unique device identifier		Sterilized by irradiation
	Country of manufacture		Individual sterile barrier system with outer protective packaging		Handle with care
	Do not resterilize		Authorized representative in the European Community		Maximum stacking
	See instructions for use.		Biological risks		Prescription use
	Keep away from sunlight		Catalog number		Model number
	Do not reuse		Batch code		Unique sterile barrier system
	Fluid path		Use by date		Date of manufacture
	This side up		Temperature limit		Fragile, handle with care
	Do not use hook		Do not use if package is damaged and see instructions for use.		Keep dry

## SPM Medicare Pvt. Ltd.

(An EN ISO 13485 Certified Company)

Corp. & Mfg. Address:

B-40, Phase-II, Noida, Gautam Buddha Nagar-201305, U.P., INDIA

Customer Care: +91-8407070718 E-mail : info@sppmedicare.com

Website : www.sppmedicare.com