



EASY BLOOD-COLLECT™

In-line arterial Blood sampling set (ABG Syringe)

INSTRUCTIONS FOR USE:

- Take pre heparinized Arterial Blood Collection Set 1ml or 3ml as required for blood sample.
- Collect 1ml or 3ml arterial/venous blood in this heparinized syringe (filling the syringe completely is very important).
- Remove air bubbles quickly (if any), and place the luer cap. Mix the blood in the syringe gently by rotating the syringe between the palms of the hands to ensure mixing of blood with heparin.
- Send the sample to the laboratory immediately. If it is not possible to send immediately then keep the syringe in a container having ice-water slurry (not with ice alone) but not for too long because the plastic syringes are partially gas permeable and this permeability increases at a lower temperature.



EXPLANATION OF SYMBOLS:

	Caution		Medical device		Non-pyrogenic
	Manufacturer		Unique device identifier		Sterilized by irradiation
	Country of manufacture		Individual sterile barrier system with outer protective packaging		Handle with care
	Do not resterilize		Authorized representative in the European Community		Maximum stacking
	See instructions for use.		Biological risks		Prescription use
	Keep away from sunlight		Catalog number		Model number
	Do not reuse		Batch code		Unique sterile barrier system
	Fluid path		Use by date		Date of manufacture
	This side up		Temperature limit		Fragile, handle with care
	Do not use hook		Do not use if package is damaged and see instructions for use.		Keep dry

SPM Medicare Pvt. Ltd.

(An EN ISO 13485 Certified Company)

Corp. & Mfg. Address:

B-40, Phase-II, Noida, Gautam Buddha Nagar-201305, U.P., INDIA

Customer Care: +91-8407070718 E-mail : info@spmmedicare.com

Website : www.spmmedicare.com