



EASY ORAL™

Oral Dispensing Syringe

INSTRUCTIONS FOR USE:

- Choose the right syringe as per the required dose.
- Fully depress the plunger of oral syringe before taking dose.
- Draw up the required dose from the bottle/container.
- Slowly push the plunger up to get rid of any air bubbles.
- Check the syringe at eye level to make sure the amount of dose.
- Place the tip of oral syringe in the mouth.
- Slowly push the plunger all the way in until the oral syringe is empty.
- Flush the oral syringe with water and dispose the syringe in container as per applicable local, state and countries law & regulations.



EXPLANATION OF SYMBOLS:

	Caution		Medical device		Non-pyrogenic
	Manufacturer		Unique device identifier		Sterilized by irradiation
	Country of manufacture		Individual sterile barrier system with outer protective packaging		Handle with care
	Do not resterilize		Authorized representative in the European Community		Maximum stacking
	See instructions for use.		Biological risks		Prescription use
	Keep away from sunlight		Catalog number		Model number
	Do not reuse		Batch code		Unique sterile barrier system
	Fluid path		Use by date		Date of manufacture
	This side up		Temperature limit		Fragile, handle with care
	Do not use hook		Do not use if package is damaged and see instructions for use.		Keep dry

SPM Medicare Pvt. Ltd.

(An EN ISO 13485 Certified Company)

Corp. & Mfg. Address:

B-40, Phase-II, Noida, Gautam Buddha Nagar-201305, U.P., INDIA

Customer Care: +91-8407070718 E-mail : info@sppmedicare.com

Website : www.sppmedicare.com