



# Easy-Vein<sup>®</sup> Control-Way

Closed IV Cannula with wings, with Extension line & with safety features

## INSTRUCTIONS FOR USE:

- Peel open the pouch prior to use.
- Ensure Vent Plug is secure & make sure clamp is not engaged.
- Twist and remove needle cover.
- Stabilize the vessel and perform cannulation.
- Pull back Needle Hub in a control & continuous motion.
- Look for Initial blood return in the catheter tubing.
- With the pad of your index finger behind the punch tab, push the catheter forward in the vessel.
- Look for the continuous blood flow up the extension tube.
- Stabilize the system and pull back until the needle shield release.
- Engage the clamp and remove the vent plug.
- Replace with a needle free connector or directly the tubing of the infusion set.



## EXPLANATION OF SYMBOLS:

	Caution		Medical device		Non-pyrogenic
	Manufacturer		Unique device identifier		Sterilized by irradiation
	Country of manufacture		Individual sterile barrier system with outer protective packaging		Handle with care
	Do not resterilize		Authorized representative in the European Community		Maximum stacking
	See instructions for use.		Biological risks		Prescription use
	Keep away from sunlight		Catalog number		Model number
	Do not reuse		Batch code		Unique sterile barrier system
	Fluid path		Use by date		Date of manufacture
	This side up		Temperature limit		Fragile, handle with care
	Do not use hook		Do not use if package is damaged and see instructions for use.		Keep dry

## SPM Medicare Pvt. Ltd.

(An EN ISO 13485 Certified Company)

Corp. & Mfg. Address:

B-40, Phase-II, Noida, Gautam Buddha Nagar-201305, U.P., INDIA

Customer Care: +91-8407070718 E-mail : info@sppmedicare.com

Website : www.sppmedicare.com